



SafeTest4

Test it, don't regret it

Colour changes for Common Drugs. For full list (over 100 per substance) go to www.SafeTest4.co.uk

Substance	Marquis	Mandelin	Mecke	Liebermann	Froehde
5-APB/6-APB	Black / Dark purple	Black / Dark purple	Black / Dark purple	Black / Dark purple	Purple
2C-B	Yellow > green or deep blue green	Green	Yellow	Very dark green	Yellow
Amphetamine	Reddish orange > reddish brown	Moderate bluish green	No Reaction	Dark green or orange	No reaction / possibly red
Butylone	Yellow	No Reaction	Brown	Greenish brown	Yellow
BZP	No Reaction	No Reaction	No Reaction	Yellow	No Reaction
Cocaine	No Reaction	Orange / Yellow	No Reaction	Yellowish or orange	Unknown
DXM	Grey > Black (flow)	No Reaction	Yellow (fast)	Purple > Black	No Reaction
Heroin	Deep purplish red	Moderate reddish brown	Deep bluish green	Black	Deep purplish red > green
lbuoprofen	No Reaction	Dark brown	Light brown	Dark reddish brown	No Reaction
Ketamine	No Reaction	Orange brown	No Reaction	Yellow (possibly faint)	No Reaction
LSD	Blotter / Extract cannot be tested	Blotter / Extract cannot be tested	Blotter / Extract cannot be tested	Blotter / Extract cannot be tested	Blotter / Extract cannot be tested
MDAI	Orange	No Reaction	Green	Green > Black	Green > Black
MDMA	Purple > Blue / Black	Blackish purple or violet > black	Green > Black	Brown / Black	Yellow / green > black
MDPV	Bright yellow	Green / Brown	Bright yellow	Yellow > green	Bright yellow
Mephedrone	No Reaction	No Reaction	No Reaction	Bright yellow	Unknown
Methylone	Bright yellow	Green / Brown or ochre	Bright yellow	Orange > Brown	Yellow
Modafinil	Yellow / Orange > brown	Brownish red	Yellow / Orange > brown	Darkening orange	Red / orange
Morphine	Deep reddish purple	Reddish brown or dark grey	Dark green/blue or green > blue	Black	Purplish red or violet > grey
PMA	No Reaction	Rust / reddish brown	No Reaction	Purple / Brown or orange	Pale green or green > brown
PMMA	No Reaction	Rust / reddish brown	No Reaction	Purple / Brown or orange	Pale green or green > brown

