



SUBSTANCE	MARQUIS	MECKE	MANDELIN	SIMON	LIEBERMANN	FROEHDE	RobaTest
	Reacts immediately. Good for weeding out bath salts and amphetamines.	Reacts immediately. Can detect DXM when no MDMA is present. Can also help identify some of the common cathinones.	Reacts immediately. Can detect PMA/PMMA when no MDMA is present. Also good for testing ketamine, MXE.	Differentiates MDMA from MDA & MDE	Reacts immediately. Can help differentiate which cathinone may be present.	Reacts immediately. Can more distinctly differentiate 5-APB or 6-APB from MDMA.	
ENTACTOGENS							
MDMA							NO REACTION
MDE							NO REACTION
MDA ("Sass")				NO REACTION			1 to 5 minutes
5-APB / 6-APB				NO REACTION			UNKNOWN
CATHINONES ("BATH SALTS")							
Methylone							UNKNOWN
Ethylone							UNKNOWN
Butylone			NO REACTION				UNKNOWN
Mephedrone (4-MMC)	NO REACTION					UNKNOWN	UNKNOWN
MDPV			NO REACTION	NO REACTION			UNKNOWN
Alpha-PVP ("Flakka")	NO REACTION	NO REACTION	NO REACTION	NO REACTION		NO REACTION	1 to 5 minutes
Methamphetamine		NO REACTION				NO REACTION (BUT POSSIBLE RED)	NO REACTION
Amphetamine		NO REACTION		NO REACTION		NO REACTION (BUT POSSIBLE RED)	1 to 5 minutes
PIPERAZINES							
BZP	NO REACTION	NO REACTION	NO REACTION	UNKNOWN		NO REACTION	NO REACTION
TFMPP	NO REACTION	NO REACTION	NO REACTION	UNKNOWN	NO REACTION	NO REACTION	NO REACTION
PMA	NO REACTION	NO REACTION		NO REACTION			SLOW
PMMA	NO REACTION	NO REACTION					NO REACTION
2C-B				NO REACTION			
2C-I			NO REACTION	NO REACTION	UNKNOWN	UNKNOWN	UNKNOWN
Ketamine	NO REACTION	NO REACTION		NO REACTION		NO REACTION	NO REACTION
Methoxetamine	NO REACTION			NO REACTION		NO REACTION	UNKNOWN
DXM			NO REACTION	NO REACTION	UNKNOWN	NO REACTION	NO REACTION
OPIATES							
Codeine				NO REACTION	UNKNOWN		NO REACTION
Morphine			NO REACTION	NO REACTION			NO REACTION
Oxycodone				NO REACTION	UNKNOWN		NO REACTION
Heroin				NO REACTION	UNKNOWN		NO REACTION
Mescaline				NO REACTION			UNKNOWN
Cocaine	NO REACTION	NO REACTION		NO REACTION		UNKNOWN	UNKNOWN
Aspirin		NO REACTION		NO REACTION			UNKNOWN
Sugar				NO REACTION			UNKNOWN

ABOUT THE KITS:

There are hundreds of drugs being sold as "Molly" or "Ecstasy." Whether you have pressed tablets, crystals or powder, there is no way to tell what drug you have without testing it first. We have tried to include the more common drugs on this color chart, but the situation is constantly changing.

Different reagents are better than others for identifying specific drugs. Our "standard set" provides the best overall sequence for identifying MDMA. MANDELIN is the best for ketamine. EHRlich's (shown on the back page) is the best for LSD. FROEHDE and LIEBERMANN can help identify some of the common cathinones.

WARNING! Reagents can only detect the presence of a substance. They cannot detect strength or purity.

ABOUT THE LIQUIDS:

MARQUIS and MECKE begin as clear liquids and gradually turn dark over a number of months. Even when the liquid is very dark inside the bottle, the reagents will still work so long as you are able to see the color reactions on a white ceramic plate.

Fresh MANDELIN starts out light orange in color and after 3-4 weeks will turn yellow and start to separate inside the bottle. Make sure and shake it before each use.

For sugar and aspirin, reactions can take up to two minutes. Aspirin reactions can vary with the brand.

WARNING! PMA & PMMA ARE EXTREMELY DANGEROUS ADULTERANTS! THEY ARE NOT RECREATIONAL DRUGS AND HAVE CAUSED HUNDREDS OF ACCIDENTAL DEATHS.